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LAVANID® - Wundgel / LAVANID® - Wundgel V+

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en INSTRUCTIONS FOR USE

LAVANID® - Wound Gel / LAVANID® - Wound Gel V+

Read the instructions for use completely before using the product. Use the product only for its intended purpose. Please note the meanings of the symbols at the end of the instructions for use.

Note for patients:

This product is used by a doctor or other healthcare professional for the treatment of wounds. Patients should be educated by a doctor or other healthcare professional before use. The type and duration of use is determined by the condition of the wound. If you notice any adverse effects, contact your doctor or health care professional.

DESCRIPTION AND INTENDED PURPOSE LAVANID® Wound Gel and LAVANID® Wound Gel V+ are sterile, ready-to-use wound gels for cleaning and moistening wounds. LAVANID® Wound Gel V+ is more viscous than LAVANID® Wound Gel. This product is suitable for acute wounds and for repeated and long-term wound treatment of chronic wounds. Chronic wounds are often covered by cell debris and dead (necrotic) tissue and may be colonised by germs. Wound coatings can lead to delayed wound healing and should be removed during wound treatment, for example by debridement. Gentle wound cleaning and wound moistening is necessary as an accompanying measure. The wound gel consists among others of an isotonic salt solution, the so-called Ringer's solution, with 0.04 % polyhexanide for preservation. The preservative polyhexanide prevents the growth of microorganisms in wound dressings and in the gel. Preservative wound dressings reduce the risk of carrying the bacteria from the wound to the patient's environment and they protect the wound from bacteria getting in from the outside. The chosen combination of Ringer's solution and polyhexanide has been well tolerated by the tissue in experimental and clinical studies. Wound treatment products preserved with polyhexanide are considered as products of choice for chronic wounds and burns.

In a clinical study on soft tissue wounds, the wound gel was well tolerated. The progressive wound healing was in line with the clinical expectations. No product-related irritation or inflammation was observed in the wound area. Patients expressed no product-related pain, burning or itching.

COMPOSITION Sodium chloride, potassium chloride, calcium chloride, water for injection, polyhexanide, glycerin, hydroxyethylcellulose.

FIELDS OF APPLICATION The wound gel is applied:

- for local moistening and cleaning of wounds
- for preservative moistening of wound dressings like cloths, tamponades and bandages

The wound gel is applied on the wound until it is sufficiently moistened. The amount of gel depends on the wound size and the dressing change interval. The wound is covered with a wound dressing to protect it from external influences. During dressing change an adherent wound dressing should be moistened with an irrigation solution like LAVANID® Wound Irrigation Solution in order to enable a gentle removal and prevent a mechanical damage of the wound surface. Residues of the wound gel should be washed off with LAVANID® Wound Irrigation Solution.

The wound gel is applied on the wound until it is sufficiently moistened. The amount of gel depends on the wound size and the dressing change interval. The wound is covered with a wound dressing to protect it from external influences. During dressing change an adherent wound dressing should be moistened with an irrigation solution like LAVANID® Wound Irrigation Solution in order to enable a gentle removal and prevent a mechanical damage of the wound surface. Residues of the wound gel should be washed off with LAVANID® Wound Irrigation Solution.

PRECAUTIONS AND WARNINGS
STERILE 1 The product is sterile as long as the container is unopened and undamaged.

- Do not use if the product shows signs of damage or if the packaging has been damaged or accidentally opened before use.
- Do not resterilise or reprocess. This can lead to critical changes in performance, safety and handling of the product.
- Do not drink or swallow the product.
- Do not mix with additives.

INTERACTIONS WITH OTHER PRODUCTS When used together with other wound treatment products, the instructions for use of these products should be checked for possible interactions with the ingredients of the wound gel.

Due to incompatibilities with the ingredient polyhexanide, the wound gel must not be used together with anionic substances.

These are, for example, substances that reduce the surface tension (anionic surfactants) or anionic gel formers or products containing PVP-iodine. In instances where there is prior contact, these substances must be rinsed liberally out of the wound.

ADVERSE EFFECTS

The ingredient polyhexanide can cause allergic reactions. In individual cases, acute reactions of the immune system (anaphylactic reactions) have been reported. Users and/or patients are encouraged to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority.

STORAGE AND SHELF LIFE

The product must not be used after the expiry date. The expiry date is indicated on the label.

Temperature limit: Store between 2 °C and 30 °C.

Store in a dry place.

Do not use if the product is stored outside the recommended storage conditions.

Fragile, handle with care.

Keep out of reach of children.

Shelf life after opening: 1 month.

Due to the preservation, a shelf-life of one month is possible after the first opening of the container. A prerequisite for this is that the gel is protected from contamination. The container must be closed immediately after each use. Gel residue around the opening should be wiped off with a sterile comp. The container opening must be protected from contamination. The container must not come into contact with the wound. Containers which have come into contact with the wound or have been contaminated in any other way must be disposed of.

INFORMATION ABOUT OPENING THE CONTAINER Information about opening the container can be found at the end of these instructions for use.

DISPOSAL INFORMATION The disposal of unused products or waste material must be carried out in accordance with specific national regulations.

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