LAVANID® - Verband

Produktbeschreibung

DE

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Product Description

EN

LAVANID®-Dressing is a sterile, moist, ready-to-use wound dressing that absorbs wound exudate and covers and moistens the wound environment. The sole purpose of the polihexanide bound in the wound dressing is to preserve the moist dressing material and provide prophylactic protection against the proliferation of bacterial and fungal pathogens in the dressing material. The moist wound dressing dissolves the biofilm, cleans the wound and creates the ideal wound climate for the formation of new granulation tissue.

Indications | Areas of Use:

Acute wounds and chronic wounds at risk of infection

- Post-operative wounds
- Ulcus cruris (lower leg ulcer)
- Superficial and deep wounds
- Diabetic foot syndrome
- Burns up to 2nd degree
- Decubitus up to grade 3

Contraindications | Warnings:

This product may only be used on skin wounds or mucosal membranes.

- Must not be used where there is hypersensitivity to one of the ingredients.
- Must not be used where bone tissue is exposed or where there is cartilage damage.
- For use on babies and pregnant women only after consultation with a doctor.

Directions for Use:

1. Carefully explore the wound and clean as per standard methods.
2. Tear the aluminium sachet where indicated to remove the **LAVANID®-Dressing** dressing. The dressing can be cut to fit the size of the wound using sterile scissors.
3. Place the **LAVANID®-Dressing** dressing on the wound to ensure the best possible contact with the wound base.
4. **LAVANID®-Dressing** is suitable for use on both sides. It is not self-adhesive and must be fixed in place using adequate products.
5. The dressing can be changed as required but should not exceed three days. However, daily changing is recommended.

Notes:

Contents only sterile provided packaging is undamaged. For single use only. For „Use by date“ see printed information on the packaging. Store in a dry place at room temperature.

For dry wounds the additional application of a hydrogel as a secondary layer to the primary polihexanide wound dressing is recommended. Ensure that the polihexanide wound dressing is in direct contact with the wound, not the secondary hydrogel layer.

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