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# LAVANOX-Serag® Wundsprügel

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## Instructions for Use:

# LAVANOX-Serag® wound spray gel

## Ready-to-use hydrogel for cleansing and moistening of wounds

### Product properties:

LAVANOX-Serag® wound spray gel can be quickly and easily applied and remains as hydrogel in the wound. It keeps the wound moist, creates a protective film and thus promotes wound healing. The application of LAVANOX-Serag® wound spray gel is non-irritating and virtually pain-free, with no restrictions to duration of product use. LAVANOX-Serag® wound spray gel is for cleaning and long-term moistening. It keeps wounds moist and forms an ideal wound healing environment, supporting the natural wound healing process. Wound odour is quickly and reliably eliminated when using LAVANOX-Serag® wound spray gel. As a multi-dose container LAVANOX-Serag® wound spray gel is preserved with NaOCl/HOCl. This allows the product to be used for within 12 weeks after first opening.

### Application:

Shake the bottle well before use! The use of a wound spray or wound irrigation solution (e.g. LAVANOX-Serag® wound spray, LAVANOX-Serag® wound irrigation solution) is recommended to remove exudate, fibrinoid residues and other contamination. Spray the wound with LAVANOX-Serag® wound spray gel by pressing the bottle pump several times at a short distance to fully moisten the wound as needed. Always generously cover the surface of the wound with LAVANOX-Serag® wound spray gel. Dress the wound with a suitable secondary dressing. LAVANOX-Serag® wound spray gel

can be used in combination with compresses, plasters and other dressing material. There are no restrictions regarding the frequency and duration of use. A slight chlorine odour indicates the activity of the preservative system.

### Application restrictions:

There are no known side effects. LAVANOX-Serag® wound spray gel should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Do not use simultaneously with products containing chlorhexidine. Do not use LAVANOX-Serag® wound spray gel on bleeding or exuding wounds. For use during pregnancy, while breast-feeding and in children under two years of age, consult your doctor or pharmacist beforehand. In children up to 12 years of age, the application should be supervised by an adult.

If there is redness, swelling or pain in the wound area or if the condition of the wound does not improve within a few days, a doctor should be consulted.

### General safety instructions:

Only apply to external dermal wounds. Do not swallow, avoid eye contact. Keep away from children. Only use intact, undamaged bottles. Only use the products prior to the expiration date indicated on the bottle. Once opened, product can only be used for a maximum of 12 weeks.

### Ingredients:

Water, sodium hypochlorite/hypochlorous acid <0.06% (electrochemically activated mineral saline solution), lithium magnesium sodium silicate

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