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PTFE PLEDGETS

en INSTRUCTIONS FOR USE

PTFE PLEDGETS

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PLEDGET IN PTFE

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PLEDGETS DE PTFE

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PTFE PLEDGETS

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PLEDGETY ZPTFE

tr KULLANIM TALİMATI

PTFE PLEJİTLER

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PTFE PLEDGETS

WARNINGS AND PRECAUTIONS

- Read the instructions for use in their entirety before using the product.
- The product should only be used by qualified and trained personnel. It is the physician's responsibility to be familiar with both appropriate surgical techniques and anatomy prior to use.
- The product is delivered sterile (sterilization method: ethylene oxide). 
- The product is intended for single use only. 
- Do not resterilize or re-use the product as critical changes in performance and safety are possible. 
- Do not use after expiration date. 
- Do not use if the product shows signs of damage (e.g. cuts or kinks) or if the outer packaging or sterile packaging has been damaged or unintentionally opened prior to use. 
- Use the product only for the intended indication.
- Please contact the manufacturer for further information.

DESCRIPTION AND INTENDED PURPOSE

PTFE Pledgets (suture buttresses) are made of non-absorbable, undyed polytetrafluoroethylene (PTFE) polymer fibres, which have been compressed to form a felt. They are available in various shapes, sizes as well as perforated and not perforated.

The main function of the pledgets is to provide durable tissue reinforcement and to form a suture buttress, especially at the heart and vessels. By providing reinforcement, the pledgets render suture pullout difficult from tissues subject to strong mechanical loading.

INSTRUCTIONS FOR USE

The pledgets are designed for use in combination with surgical suture material. In the choice and use of the pledgets, due account should be taken of the patient's condition, the experience of the physician and the surgical technique.

CONTRAINDICATIONS

The device must not be used in patients with allergies to any of the product components.

RESTRICTIONS AND LIMITATIONS OF USE / NOTES/ INTERACTIONS

Improper handling with surgical instruments or contact with sharp edges should be avoided to prevent fraying of fibres or other mechanical damage. PTFE Pledgets should not be exposed to laser beams. Penetration of pledgets outside the prepunched holes may increase needle insertion force.

ADVERSE REACTIONS

The following adverse reactions can occur in association with correct use of the device:

local irritation, inflammatory reactions to the foreign body; formation of foreign-body giant cells, suture fistulas or granulomas.

Users and/or patients are encouraged to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority.

STORAGE CONDITIONS

The storage conditions apply as indicated on the label.

DISPOSAL INFORMATION

Disposal of unused products or waste material with particular care due to the potential risk of contamination and/or injury. In all other cases observe the specific national regulations.

MEANING OF THE SYMBOLS

Symbols are explained at the end of the instructions for use.

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**SERAG-WIESSNER GmbH & Co. KG**