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LAVANOX-Serag® Wundspüllösung

Produktigenschaften:

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SERAG-WIESSNER GmbH & Co. KG
Zum Kugelfang 8-12
95119 Naila/Germany

Instructions for Use:

LAVANOX-Serag® wound irrigation solution

Product properties:

LAVANOX-Serag® wound irrigation solution is used for cleaning, moistening, mechanical decontamination and infection prevention in the acute and chronic wound. The application of LAVANOX-Serag® wound irrigation solution is non-irritating. LAVANOX-Serag® wound irrigation solution is used for cleaning the wound by helping to remove biofilms and fibrinoid residues. In addition LAVANOX-Serag® wound irrigation solution can aid the gentle removal of dressings. As a multi-dose container LAVANOX-Serag® wound irrigation solution is preserved with NaOCl/HOCl. This allows the product to be used for within 12 weeks after first opening. Wound odour is quickly and reliably eliminated when using LAVANOX-Serag® wound irrigation solution.

Application:

The wound should first be cleaned to remove dirt, blood and any residues. The affected area is thoroughly cleaned using generous amounts of LAVANOX-Serag® wound irrigation solution. In the event of major contamination, the application can be repeated several times. Once the cleaning procedure is finished, a secondary cleaning solution is not required (i.e. any residual LAVANOX-Serag® wound irrigation solution can be left in the wound). Dressings or bandages that are difficult to remove are soaked in LAVANOX-Serag® wound irrigation solution until they are wet. After a short waiting period, they can then be carefully removed virtually pain-free. If needed, LAVANOX-Serag® wound spray gel can be applied. There are no restrictions regard-

ing the frequency and duration of use. A slight chlorine odour indicates the activity of the preservative system.

Application restrictions:

There are no known side effects. LAVANOX-Serag® wound irrigation solution should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings Do not use simultaneously with products containing chlorhexidine. During pregnancy and whilst breastfeeding LAVANOX-Serag® wound irrigation solution should be used with caution.

General safety instructions:

Not intended for infusion or injection!

Only apply to external dermal wounds, do not swallow, avoid eye contact. Keep away from children. Only use intact, undamaged bottles.

Only use the products prior to the expiration date indicated on the bottle. Once opened, product can only be used for a maximum of 12 weeks.

Ingredients:

Water, sodium hypochlorite/hypochlorous acid <0.08% (electrochemically activated mineral saline solution)

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