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LAVANOX-Serag® Wundspray

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LAVANOX-Serag® wound spray

Product properties:

LAVANOX-Serag® wound spray is used for cleaning, moistening, antimicrobial decontamination and infection prevention in the acute and chronic wound. The application of LAVANOX-Serag® wound spray is non-irritating. LAVANOX-Serag® wound spray is used for cleaning the wound by helping to remove biofilms and fibrinoid residues. In addition LAVANOX-Serag® wound spray can aid the gentle removal of dressings. Wound odour is quickly and reliably eliminated when using LAVANOX-Serag® wound spray. As a multi-dose container LAVANOX-Serag® wound spray is preserved with NaOCl/HOCl. This allows the product to be used for within 12 weeks after first opening.

Application:

The wound should first be cleaned to remove dirt, blood and any residues. The affected area is sprayed with LAVANOX-Serag® wound spray by pressing the pump on the bottle several times, at a short distance to fully moisten the wound. If necessary, for mechanical cleaning and debridement, in the event of major contamination, the application can be repeated several times. Once the cleaning procedure is finished, a secondary cleaning solution is not required (i.e. any residual LAVANOX-Serag® wound spray can be left in the wound). Dressings or bandages that are difficult to remove are sprayed with LAVANOX-Serag® wound spray until they are wet. After a short waiting period, they can then be carefully removed virtually painfree. If needed, LAVANOX-Serag® wound spray gel can be applied. There are no restrictions regarding the frequency and duration of use. A slight chlo-

rine odour indicates the activity of the preservative system.

Application restrictions:

There are no known side effects of using LAVANOX-Serag® wound spray. LAVANOX-Serag® wound spray should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Do not use simultaneously with products containing chlorhexidine. During pregnancy and when breastfeeding LAVANOX-Serag® wound spray should be used with caution.

General safety instructions:

Not intended for infusion or injection! Only apply to external dermal wounds. Do not swallow or spray in eyes. Keep away from children. Only use intact, undamaged bottles. Only use the products prior to the expiration date indicated on the bottle. Once opened, product can only be used for a maximum of 12 weeks.

Ingredients:

Water, sodium hypochlorite/hypochlorous acid <0.08% (electrochemically activated mineral saline solution)

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