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<b>en</b> <b>INSTRUCTIONS FOR USE</b>
<b>LAVANID® - Wound Gel V+</b>

**DESCRIPTION AND PRODUCT PROFILE**

The surface of chronic wounds is generally made up of devitalized and necrotic tissue, and is also colonised with bacteria. For wound treatment to be successful the wound surface needs to be removed, e.g. through surgical debridement. Wound irrigation and moistening are a necessary part of this process. Chronic wounds exhibit dysfunctional wound healing processes. This is why treatment with preparations that are very well tolerated by the tissues is indicated.

LAVANID® Wound Gel V+ is a sterile, preserved Ringer’s-solution-based wound gel which stands out due to its very good tissue tolerance. Pol-hexanide 0.04% is used as a preservative. Polyhexanide is evaluated as the agent of choice for chronic wounds and burns. In addition to cleaning and moistening wounds, LAVANID® Wound Gel V+ is also appropriate for preservative dressings, e.g. for wounds colonised with multiresistant bacteria. Preservative dressings reduce the risk of carrying the bacteria from the wound to the patient’s environment and they protect the wound from bacteria getting in from the outside.

**COMPOSITION**  
Ringer’s solution (isotonic electrolyte solution made up of sodium chloride, potassium chloride, calcium chloride 2 H<sub>2</sub>O, water for injection), glycerin, macrogol, hydroxyethylcellulose, polyhexanide 0.04%.

**INDICATIONS**  
LAVANID® Wound Gel V+ is used to moisten and clean wounds and to moisten bandages and wound dressings with preservative.

**TISSUE TOLERANCE**  
Experimental and clinical studies have shown that the combination of Ringer’s solution with 0.04% polyhexanide is very well tolerated by tissue. Clinical trials have shown that it does not negatively affect the formation of healthy new tissue. There are no local signs of irritant conditions. Use of this product is pain-free for patients. Because it is so well tolerated by tissues LAVANID® Wound Gel V+ is suitable for repeated and long-term wound treatment.

**INFORMATION ABOUT SHELF-LIFE**  
The expiry date can be found on the container. Do not use after the expiry date!

LAVANID® Wound Gel V+ is sterile as long as the packaging is unopened and undamaged. Although it is no longer sterile after the container has been opened, because it contains a preservative it can be used up to one month after opening under certain conditions. These conditions are that the contents be protected from contamination. The container must not come in contact with the wound. The opening should be capped immediately after each use. Gel residue around the opening should be wiped off with a sterile compress.

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