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# LAVANOX® Wundsprügel

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## Instructions for Use:

# LAVANOX® wound spray gel

## Ready-to-use hydrogel for cleansing and moistening of wounds

### Product properties:

LAVANOX® wound spray gel can be quickly and easily applied and remains as hydrogel in the wound. It keeps the wound moist, creates a protective film and thus promotes wound healing.

The application of LAVANOX® wound spray gel is non-irritating and virtually pain-free, with no restrictions to duration of product use.

LAVANOX® wound spray gel is for cleaning and long-term moistening. It keeps wounds moist and forms an ideal wound healing environment, supporting the natural wound healing process.

The preservative (sodium hypochlorite) contained in LAVANOX® wound spray gel decontaminates a broad spectrum of bacteria (incl. pseudomonas aeruginosa, MRSA/MRE), viruses and fungi. LAVANOX® can also be used in the area of wounds with exposed cartilage, tendons, ligaments or bones. Wound odour is quickly and reliably eliminated when using LAVANOX® wound spray gel.

### Application:

**Shake the bottle well before use!**

The use of a wound spray or wound irrigation solution (e.g.

LAVANOX® wound spray, LAVANOX® wound irrigation solution) is recommended to remove exudate, fibrinoid residues and other contamination. Spray the wound with LAVANOX® wound spray gel by pressing the bottle pump several times at a short distance to fully moisten the wound as needed.

Always generously cover the surface of the wound with LAVANOX® wound spray gel. Dress the wound with a suitable secondary dressing.

LAVANOX® wound spray gel can be used in combination with compresses, plasters and other dressing material. There are no restrictions regarding the frequency and duration of use.

A slight chlorine odour indicates the product is working.

### Application restrictions:

There are no known side effects. LAVANOX® wound spray gel should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Interactions with other substances are not expected. During pregnancy and whilst breastfeeding LAVANOX® wound spray gel should be used with caution. Do not use LAVANOX® wound spray gel on bleeding or exuding wounds.

### General safety instructions:

Only apply to external dermal wounds. Do not swallow, avoid eye contact. Keep away from children. Only use intact, undamaged bottles.

Only use the products prior to the expiration date indicated on the bottle. Once opened, product can only be used for a maximum of 12 weeks.

### Ingredients:

Water, sodium hypochlorite <0.06% (electrochemically activated mineral saline solution), lithium magnesium sodium silicate

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