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LAVANOX-Serag® Wundspüllösung 1000ml

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Instructions for Use:

LAVANOX-Serag® wound irrigation solution 1000ml

Product properties:

LAVANOX-Serag® wound irrigation solution is used for cleaning, moistening, mechanical decontamination and infection prevention in the acute and chronic wound. The application of LAVANOX-Serag® wound irrigation solution is non-irritating. LAVANOX-Serag® wound irrigation solution is used for cleaning the wound by helping to remove biofilms and fibrinoid residues. In addition LAVANOX-Serag® wound irrigation solution can aid the gentle removal of dressings. Suitable for use in negative pressure wound therapy (NPWT). As a multi-dose container LAVANOX-Serag® wound irrigation solution is preserved with NaOCl/HOCl. This allows the product to be used for within 12 weeks after first opening. Wound odour is quickly and reliably eliminated when using LAVANOX-Serag® wound irrigation solution.

Application:

The wound should first be cleaned to remove dirt, blood and any residues. The affected area is thoroughly cleaned using generous amounts of LAVANOX-Serag® wound irrigation solution. In the event of major contamination, the application can be repeated several times. Once the cleaning procedure is finished, a secondary cleaning so-

lution is not required (i.e. any residual LAVANOX-Serag® wound irrigation solution can be left in the wound). Dressings or bandages that are difficult to remove are soaked in LAVANOX-Serag® wound irrigation solution until they are wet. After a short waiting period, they can then be carefully removed virtually pain-free. If needed, LAVANOX-Serag® wound gel or wound spray gel can be applied. There are no restrictions regarding the frequency and duration of use. A slight chlorine odour indicates the activity of the preservative system.

Application restrictions:

There are no known side effects. LAVANOX-Serag® wound irrigation solution should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings Do not use simultaneously with products containing chlorhexidine. For use during pregnancy, while breast-feeding and in children under two years of age, consult your doctor or pharmacist beforehand. In children up to 12 years of age, the application should be supervised by an adult. If there is redness, swelling or pain in the wound area or if the condition of the wound does not improve within a few days, a doctor should be consulted.

General safety instructions:

Not intended for infusion or injection! Only apply to external dermal wounds, do not swallow, avoid eye contact. Keep away from children. Only use intact, undamaged bottles. Only use the products prior to the expiration date indicated on the bottle. Once opened, product can only be used for a maximum of 12 weeks.

Ingredients:

Water, sodium hypochlorite/hypochlorous acid <0.08% (electrochemically activated mineral

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