

Product range

# ENDO-NAHT MSD NEY

Surgical  
sutures



**SERAG**  
**WIESSNER**



## Information

### ENDO-NAHT MSD NEY

The suturing system consists of a suture guiding tube and a needle-thread combination (SERAFast® violet USP 3/0 / EP2) with a special double-curved needle.

The guiding tube is made of surgical steel and has an outer diameter of 2 mm and a structured handle. A special feature is the fixation slot, into which the specially formed needle can be locked in place when necessary.

### MSD NEY special needles

The special MSD NEY needles DSSB-24 have a cutting needle body with an extra slender bevel and a 3/8 double curve with a 24 mm curve length.

They are available in a left- and right-turning shape.

### MSD NEY knot slider


The knot (e.g. Roeder knot) is tied using the extracorporeal technique and placed at the closure site with the aid of the special MSD NEY knot slider.



## Suture material

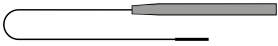




SERAFAST® is the right choice for indications with a short wound healing time, when all the advantages of monofilament sutures count.

**Material**  POLYGLYKOLIC ACID CAPROLACTONE

**Suture symbol**  violet, monofilament, absorbable

**Suture size** USP 3/0  
EP 2

**Absorption profile** At least 50% residual tensile strength after 7 days,  
completely absorbed after approx. 90 days

Symbol	Product name	Description/Size	Quantity	Art.-Number
  	ENDO-NAHT MSD NEY	DSSB-24-R SERAFAST® violet USP 3/0 (EP2), 1 x 1, 20 m	3 pcs	<b>JZ2048N1</b>
		DSSB-24-L SERAFAST® violet USP 3/0 (EP2), 1 x 1, 20 m	3 pcs	<b>JZ2048N2</b>
 	KNOT SLIDER		1 pc	<b>22</b>

The risks that may be encountered, even with the correct use of the products, are given in the Instructions for Use. These are available on the SERAG-WIESSNER website at [www.serag-wiessner.de](http://www.serag-wiessner.de).

In collaboration with:

Dr Jörg Neymeyer

Consultant of Urology

Consultant of Gynaecology and  
Obstetrics

Head of the Urogynaecology Section


Department for Urology University  
Hospital Charité, Berlin




SERAG-WIESSNER GmbH & Co. KG


Zum Kugelfang 8 - 12


95119 Naila/Germany

 + 49 9282 937-0


 + 49 9282 937-9369

*Export Department:*

 + 49 9282 937-230

 + 49 9282 937-9785

 [info@serag-wiessner.de](mailto:info@serag-wiessner.de)

 [www.serag-wiessner.de](http://www.serag-wiessner.de)

