

Further development of the proven instrument

SERAPRO® VR

Instruments



For retropublic use

Easier handling
right on course



Advantages of SERAPRO® VR

- reusable surgical steel instrument, **MADE IN GERMANY**
- easy to dismantle and clean
- secure locking element ensures safe instrument interchange
- expands surgical sets as desired

- combines a range of different tunnelers (length, tip, curvature, strength) with a single handle
- enhances and expands the tried-and-tested SERASIS® implantation technique by continued retropubic tunnelling in the same direction

Preparing the instrument



1. Insert the locking element



2. Rotate the locking element through 90° (secure)



3. Push up the locking element - insert tunneller



4. Press the locking element down

Use



1. Create tunnel behind the symphysis



2. Push up the locking element - release the tunneller



3. Insert SERASIS® tape into the eye with overlap



4. Continue retropubic passage in the same direction



SERAG-WIESSNER GmbH & Co. KG
Zum Kugelfang 8-12
95119 Naila/Germany

+ 49 9282 937-230

+ 49 9282 937-9785

info@serag-wiessner.de

www.serag-wiessner.de