LAVANID® - Verband

Produktbeschreibung

DE

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Product Description

EN

LAVANID®-Dressing is a sterile moist - ready-to-use - wound dressing, impregnated with **PHMB* Antimicrobial Complex**. This solution decontaminates and serves as a prophylaxis against infection with micro-organisms. It also provides prophylaxis against bacterial and fungal pathogens in both acute and chronic wounds. The effective combination of the ingredients of **LAVANID®-Dressing** dissolves the biofilm, cleans the wound and creates the ideal wound climate for the formulation of new granulation tissue. (*Polyhexamethylene biguanide)

Indications | Areas of Use:

Acute wounds and chronic wounds at risk of infection

- Post-operative wounds
- Ulcus cruris (lower leg ulcer)
- Superficial and deep wounds
- Diabetic foot syndrome
- Burns up to 2nd degree
- Decubitus up to grade 3

Contraindications | Warnings:

This product may only be used on skin wounds or mucosal membranes.

- Must not be used where there is hypersensitivity to one of the ingredients.
- Must not be used where bone tissue is exposed or where there is cartilage damage.
- For use on babies and pregnant women only after consultation with a doctor.

Directions for Use:

1. Carefully explore the wound and clean as per standard methods.
2. Tear the aluminium sachet where indicated to remove the **LAVANID®-Dressing** dressing. The dressing can be cut to fit the size of the wound using sterile scissors.
3. Place the **LAVANID®-Dressing** dressing on the wound to ensure the best possible contact with the wound base.
4. **LAVANID®-Dressing** is suitable for use on both sides. It is not self-adhesive and must be fixed in place using adequate products.
5. The dressing can be changed as required but should not exceed three days. However, daily changing is recommended.

Notes:

Contents only sterile provided packaging is undamaged. For single use only. For „Use by date“ see printed information on the packaging. Store in a dry place at room temperature.

For dry wounds the additional application of a hydrogel as a secondary layer to the primary PHMB wound dressing is recommended. Ensure that the PHMB wound dressing is in direct contact with the wound, not the secondary hydrogel layer.

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